User Manual

ACRUM

2234A.3.A26

VERSION 1.24

Weelkome!

Thank you for having acquired a WEELKO product and for the trust shown in us.

This treatment table has been designed to fulfill the most demanding work requirements of those professionals working in massage or physiotherapy fields. It can also be used for recovery treatments or examination. This table should only be used by trained professionals. Although the treatment bed is designed for use in multiple treatments, it should never be used for any purpose than its intended design limitations.

The company reserves the right to modify this manual with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail to info@weelko.com informing us so that we can correct them.

Use of the manual

Before using the table please read the instruction manual carefully. It is important to keep this manual and store it in a safe place so that it can be consulted at any time.

If you follow these instructions carefully and take into account the safety measures detailed, you will avoid any problems when using the table/bed.

Some texts and/or illustrations may not belong to your table, since they are general and they may depend on your product choice at the purchase time.

Our company reserves the right to improve the design of the product and modify them without prior notice.

Warranty

The warranty period is effective from the date of purchase of the product. It covers all the components of the table or beauty bed as long as the product is used under normal conditions and correct use. Please contact your point of sale to get further information on warranty terms and conditions. The warranty does not cover defects or malfunction caused by:

- Misuse, abuse or improper maintenance of the table and its upholstery;
- Non-observance of the precautions and warnings stated in this instruction manual;
- Lack of maintenance and maintenance or repair tasks carried out by non-qualified service personnel who are unauthorized by the manufacturer;
- Problems or breakdowns that may appear after non-authorized repairs performed by third parties, specially non-authorized personnel;
- Wrong and faulty electrical installations or which do not fulfil the legal requirements;
- The company is not liable for incidental or consequential damage to property or business.

EC Declaration of conformity

This product is produced and marked in conformity with the directives hereby stated :

Low voltage Directive : 2014/35/CE
Electromagnetic compatibility : 2014/30/CE

Waste management

The symbol of the crossed-out container means that it is legally required to dispose of the product separately from unsorted urban waste. It means that it must be disposed of separately from household waste.

The proper disposal of these products contributes to saving valuable resources and prevents negative effects on human health and the environment if not discarded properly.

For their proper treatment, recovery, and recycling, take these products to designated collection points or consult the local laws of your country.

Safety instructions



- Place the treatment table or beauty bed in a space with free access from the 4 sides. Do not place the table against the wall or against any obstacles.
- Place the table on a stable and horizontal surface. Level it if necessary with the levelling feet under the table.
- Do not use the table in a high-humidity environments. If the table gets wet accidentally, unplug the table immediately.
- Check if the mains voltage and frequency are in accordance with the information on the typeshield. If not, contact your supplier before connecting the device.
- Make sure that the unit is correctly plugged into an earth intake, following the local and national requirements and standards.
- Pay special attention that the mains cable does not get pinched under the feet of the table or its base frame.
- If the power cord or plug is damaged, please contact the distributor, service center or experienced technicians for repairing or replacement.
- Do not leave children unattended near the table.
- Unless being monitored or instructed by personnel who can be responsible for their personal safety, the people who have physical obstacles, tactile disorders, mental disorders, lack of experience, common sense or children, are not allowed to use the equipment.

During treatment

- Don't let anybody, except the operator and the patient, in the treatment room. Patients are not to be left unattended,
- The table is not intended to transport patients. Therefore it cannot be used for this purpose.
- Do not sit on the head section or end of the leg section of the table. The head section and armrests are not suitable to sit on. RISK OF DAMAGE!
- Never leave the table on its wheels during the treatment.
- Do not exceed the maximum load capacity of the table.
- Place the table on a stable and horizontal surface. Level it if necessary with the levelling feet under the table.
- Guide the backrest section with the other hand when adjusting the backrest by gas levers.
- Ensure that neither the patient nor the operator can get any part of the body or clothes trapped between the moving parts of the mechanism when the table is being raised or lowered.

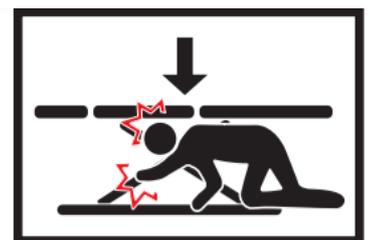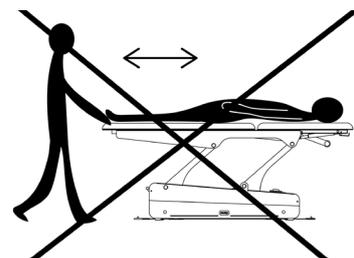Product description

High-performance table specially conceived for physiotherapy treatments. Equipped with 3 motors that control height, backrest and central section inclination. Double armrest system: the fixed ones are adjustable by gas lift and the lateral ones are removable. Headrest with breather hole. Extra postural cushion provided. Curved base with two wheels to facilitate its mobility. White PU upholstery of high quality that eases cleaning. Paper roll holder included.

Unboxing


Open the top of the packaging carefully. Do not use any sharp-edged items which can damage the upholstery.

Should the table/bed need to be moved or lifted (to take it out from its packaging) then a minimum of 2 people are required. The table must only be lifted by the metal frame near the wheels.

When you receive the treatment table or bed, make sure that the packaging contains all the elements necessary and that there are no missing parts.


The table comes ready to use.

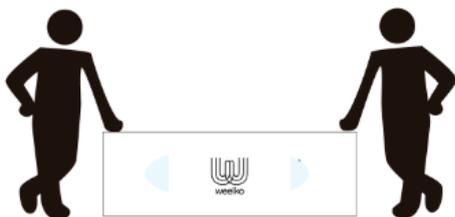Installation

Place the table in the final location. It is not necessary to screw the table to the floor to secure it. The table design already takes into account its structural stability. Make sure it is placed on a flat and firm surface to prevent any incidents. If the table wobbles or moves due to an uneven floor, adjust the 4 leveling feet and nuts with a hex key (included in the bottom panel). Level it until it is stable and plug it into the power source.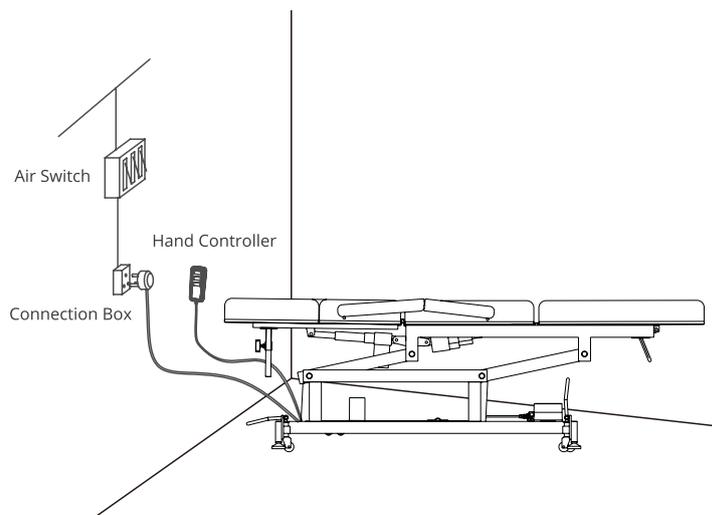

Operation


Once the table has been unpacked and placed at the final location, the table is ready for use.

Adjusting the 3 motors of the table

The table is standardly equipped with a handcontroller: It allows adjusting the 3 engines:

- 1 Move the backrest UP
- 2 Move the backrest DOWN
- 3 Lift the table
- 4 Move the table DOWN
- 5 Move the middle section UP
- 6 Move the middle section DOWN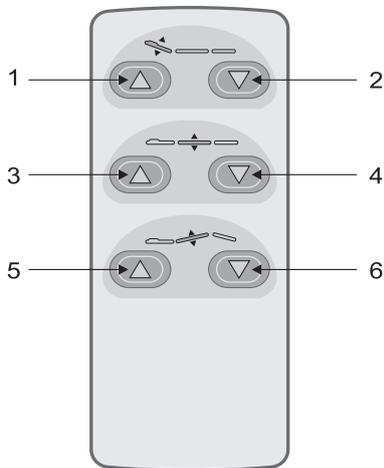


3-section head

The side armrests of model 2234A have a wheel with a screw. When the wheel is loosened, it allows you to lower and raise the armrest as well as to rotate it inward and outward. To fix them at place, tighten the wheel again until the mechanism is securely fixed. Both armrests operate in the same way.

Option: foot controller

Ref. FC003 (footcontroller)

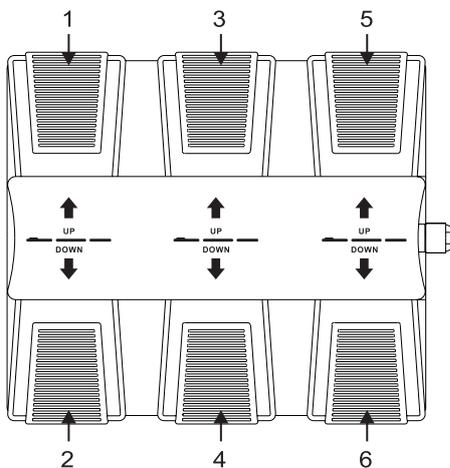Side armrests (middle section)

The table is equipped with two extra armrests placed at each side of the middle section. The table arrives with the side armrests apart. You must mount them. The metal rods of each armrest must be fit into the holes (see image 2; backrest and middle section).

Image 2

Paper roll holder

The table is equipped as standard with a paper roll holder placed at the end of the table (at the foot part under the top structure). Suitable for rolls of 50 cm width.

Wheel system to move the table

The metal bar used to lock and unlock the wheels is placed at both ends of the table base. See image below.

Simply push the bar downwards and the wheels will be unlocked. Now you can easily move the table. To lock the wheels and set the table fix on the floor, proceed the other way round and release the metal bar with your foot.

LEVER DOWN= Wheels ACTIVATED and ON
LEVER UP= Wheels RETRACTED and OFF.

The table has 4 wheels turning 360°.

The table is not designed for patient transportation. But they are helpful when moving the table from one place to another or for cleaning tasks.

Maintenance & Cleaning

Upholstery daily cleaning:

- Before performing cleaning and maintenance tasks on the treatment table, unplug it from the power source.
- To ensure an adequate level of hygiene, we recommend cleaning the upholstery after each use.
- Clean at least once a week to maintain the coating clean and prevent accumulation of dirt and contaminants. This will extend the lifespan of your upholstery.
- Clean the entire surface using neutral PH liquid soap and water in circular motions. To do so, use a white clean cloth soaked in the cleansing solution and wring out the wet cloth until all water is removed.
- Avoid excessively wetting the surface of the treatment table. Use a well-wrung cloth.
- Dry the surface using a white and clean cloth.
- Never use solvents, detergents and/or cleaning products that are not pH neutral.
- Never use cleaning tools made of plastic, abrasive sponges or hard-bristle brushes.
- Important: Lacquers, strong cleaners, xylene-based solutions, acetone or MEK cause immediate damage and contribute to the deterioration of the material. The use of such cleaners is at the owner's risk.

Table frame cleaning:

- Remove dust with a white cloth.
- Use alcohol-free cleaning wipes to clean the structure or simply use a white cotton cloth soaked in a mix of water with neutral diluted soap and rinse. Dry with a clean cloth.

Maintenance

Weelko tables have been designed for an intensive use during long time thanks to their stability, reliability and durability. However, we advise to keep the table in perfect conditions and make a correct use of it. Unplug the table if you will not use it for a long period of time.

Technical service




Only a technician authorised by Weelko is allowed to perform any kind of repairs on the bed.

Please contact your distributor or point of sales for further information on technical assistance.

Technical Specifications

ACRUM reference	2234A.A26
Packaging dimensions (cm)	204 x 75 x 60
Base dimensions (cm)	120 x 63
Gross weight (kg)	100
Net weight (kg)	85
Motors	
Motors	3 (height, backrest and middle section)
Height (cm)	55 / 95
Backrest angle (°)	-20° ~ + 80°
Middle section inclination (°)	0° ~ + 20°
Total length (cm)	200
Total width (cm)	73 (92 cm with side armrests)
Thickness of upholstery (cm)	10
Density	25 kg/m ³
Upholstery material	White PU
Electrical	
Voltage	220-240 V
Frequency	50/60Hz
Maximum power	300W
Duty cycle	Max. 2Min ON; 18min OFF
Max. lifting capacity:	180 kg
Controllers	
Hand controller	HC003 (standard)
Foot controller	FC003 (option)

Product dimensions

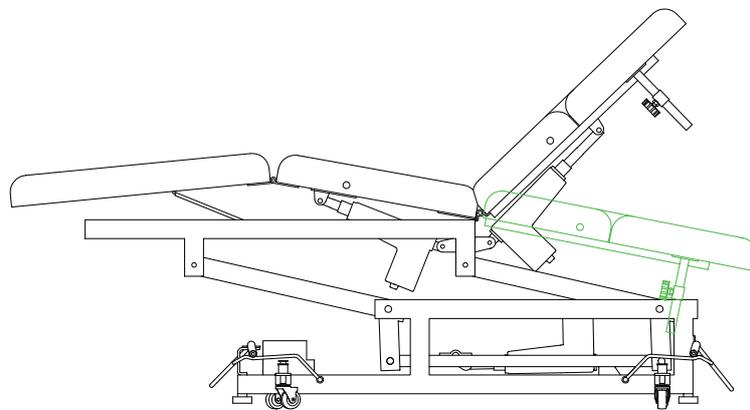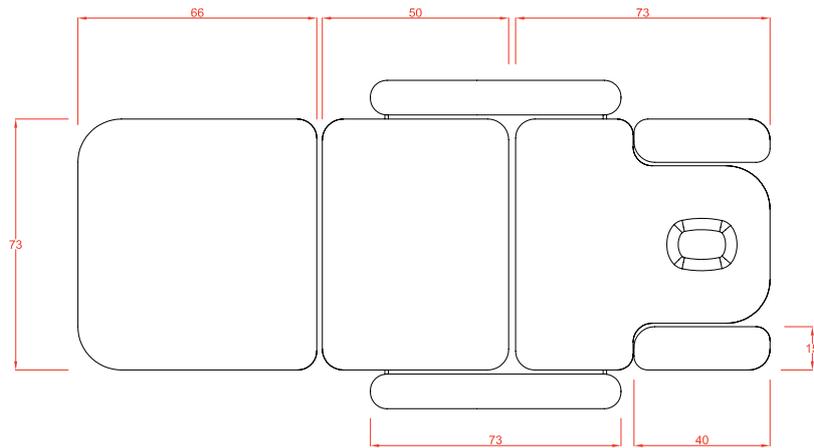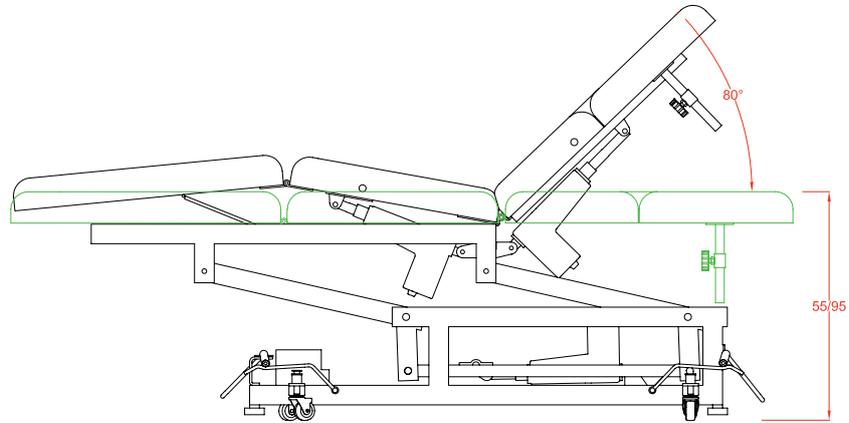Pol. Ind. La Vallan
Camí de Garreta, 92
08600 Berga
Barcelona - Spain
T. +34 93 824 90 20
E. info@weelko.com

Our products are subject to
continuous technical improvements.

Weelko reserves the right to
change or modify this manual and
its contents without prior notice.

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