# Instructions Manual

# CERVIC

2241.1.A26  
2241.1.A66

VERSION 1.24

---

## Weelkome!

Thank you for having acquired a WEELKO product and for the trust shown in us.

This treatment table has been designed to fulfill the most demanding work requirements of those professionals working in massage or physiotherapy fields. It can also be used for recovery treatments or examination. This table should only be used by trained professionals. Although the treatment bed is designed for use in multiple treatments, it should never be used for any purpose than its intended design limitations.

The company reserves the right to modify this manual with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail to [info@weelko.com](mailto:info@weelko.com) informing us so that we can correct them.

---

## Use of the manual

Before using the table please read the instruction manual carefully. It is important to keep this manual and store it in a safe place so that it can be consulted at any time.

If you follow these instructions carefully and take into account the safety measures detailed, you will avoid any problems when using the table/bed.

Some texts and/or illustrations may not belong to your table, since they are general and they may depend on your product choice at the purchase time.

Our company reserves the right to improve the design of the product and modify them without prior notice.

---

## Warranty

The warranty period is effective from the date of purchase of the product. It covers all the components of the table or beauty bed as long as the product is used under normal conditions and correct use. Please contact your point of sale to get further information on warranty terms and conditions. The warranty does not cover defects or malfunction caused by:

- Misuse, abuse or improper maintenance of the table and its upholstery;
- Non-observance of the precautions and warnings stated in this instruction manual;
- Lack of maintenance and maintenance or repair tasks carried out by non-qualified service personnel who are unauthorized by the manufacturer;
- Problems or breakdowns that may appear after non-authorized repairs performed by third parties, specially non-authorized personnel;
- Wrong and faulty electrical installations or which do not fulfil the legal requirements;
- The company is not liable for incidental or consequential damage to property or business.

---

## EC Declaration of conformity

This product is produced and marked in conformity with the directives hereby stated :

Low voltage Directive : 2014/35/CE  
Electromagnetic compatibility : 2014/30/CE

---

## Waste management

The symbol of the crossed-out container means that it is legally required to dispose of the product separately from unsorted urban waste. It means that it must be disposed of separately from household waste.

The proper disposal of these products contributes to saving valuable resources and prevents negative effects on human health and the environment if not discarded properly.

For their proper treatment, recovery, and recycling, take these products to designated collection points or consult the local laws of your country.

# Safety instructions



- Place the treatment table or beauty bed in a space with free access from the 4 sides. Do not place the table against the wall or against any obstacles.
- Place the table on a stable and horizontal surface. Level it if necessary with the levelling feet under the table.
- Do not use the table in a high-humidity environments. If the table gets wet accidentally, unplug the table immediately.
- Check if the mains voltage and frequency are in accordance with the information on the typeshield. If not, contact your supplier before connecting the device.
- Make sure that the unit is correctly plugged into an earth intake, following the local and national requirements and standards.
- Pay special attention that the mains cable does not get pinched under the feet of the table or its base frame.
- If the power cord or plug is damaged, please contact the distributor, service center or experienced technicians for repairing or replacement.
- Do not leave children unattended near the table.
- Unless being monitored or instructed by personnel who can be responsible for their personal safety, the people who have physical obstacles, tactile disorders, mental disorders, lack of experience, common sense or children, are not allowed to use the equipment.

# During treatment

- Don't let anybody, except the operator and the patient, in the treatment room. Patients are not to be left unattended,
- The table is not intended to transport patients. Therefore it cannot be used for this purpose.
- Do not sit on the head section or end of the leg section of the table. The head section and armrests are not suitable to sit on. RISK OF DAMAGE!
- Never leave the table on its wheels during the treatment.
- Do not exceed the maximum load capacity of the table.
- Place the table on a stable and horizontal surface. Level it if necessary with the levelling feet under the table.
- Guide the backrest section with the other hand when adjusting the backrest by gas levers.
- Ensure that neither the patient nor the operator can get any part of the body or clothes trapped between the moving parts of the mechanism when the table is being raised or lowered.

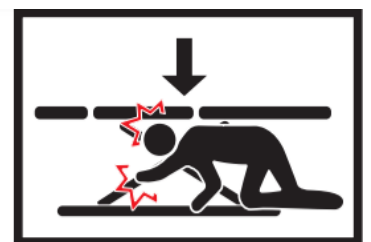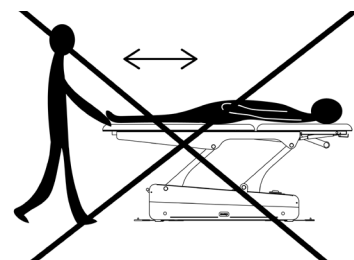# Product description

2-section table with a minimalist design. Equipped with 1 motor that controls the table height. Backrest adjustable by gas lift. Sturdy curved metallic base with 2 wheels to facilitate its mobility. White or dark grey PU upholstery of high quality that eases cleaning. Paper roll holder included.

# Unboxing

Open the top of the packaging carefully. Do not use any sharp-edged items which can damage the upholstery.

Should the table/bed need to be moved or lifted ( to take it out from its packaging) then a minimum of 2 people are required. The table must only be lifted by the metal frame near the wheels.

When you receive the treatment table or bed, make sure that the packaging contains all the elements necessary and that there are no missing parts.


The table comes ready to use.

# Installation

Place the table in the final location. It is not necessary to screw the table to the floor to secure it. The table design already takes into account its structural stability. Make sure it is placed on a flat and firm surface to prevent any incidents. Once in place, plug it into the power supply socket.

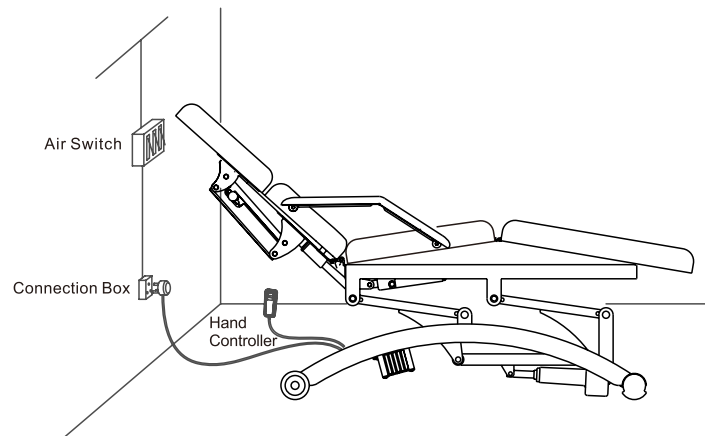
# Operation

Once the table has been unpacked and placed at the final location, the table is ready for use.


## Adjusting the height of the table

The table comes equipped as standard with a hand-held control that allows you to adjust the height of the table. Press 1 to raise it and 2 to lower it.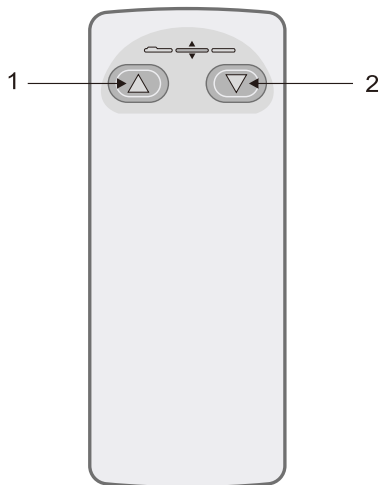

## Options for the height adjustment

Ref. FC001 (Foot controller)

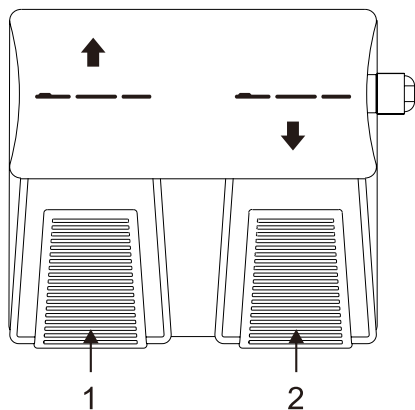## Adjusting the backrest

The table is equipped with a level that blocks and unblocks the gas lift. Keep the gas lift pressed and move the backrest at the desired inclination, then release the lever to keep it fixed.

## Wheel system

The 2241 massage table has 2 one-way wheels at one end of the table (foot part). In order to move it, grab the table by the metal structure under the backrest, lift it a bit and move it forward as if it were a hand truck. Don't walk backwards.

# Maintenance & Cleaning

## Upholstery daily cleaning:

- Before performing cleaning and maintenance tasks on the treatment table, unplug it from the power source.
- To ensure an adequate level of hygiene, we recommend cleaning the upholstery after each use.
- Clean at least once a week to maintain the coating clean and prevent accumulation of dirt and contaminants. This will extend the lifespan of your upholstery.
- Clean the entire surface using neutral PH liquid soap and water in circular motions. To do so, use a white clean cloth soaked in the cleansing solution and wring out the wet cloth until all water is removed.
- Avoid excessively wetting the surface of the treatment table. Use a well-wrung cloth.
- Dry the surface using a white and clean cloth.
- Never use solvents, detergents and/or cleaning products that are not pH neutral.
- Never use cleaning tools made of plastic, abrasive sponges or hard-bristle brushes.
- Important: Lacquers, strong cleaners, xylene-based solutions, acetone or MEK cause immediate damage and contribute to the deterioration of the material. The use of such cleaners is at the owner's risk.

## Table frame cleaning:

- Remove dust with a white cloth.
- Use alcohol-free cleaning wipes to clean the structure or simply use a white cotton cloth soaked in a mix of water with neutral diluted soap and rinse. Dry with a clean cloth.

## Maintenance

Weelko tables have been designed for an intensive use during long time thanks to their stability, reliability and durability. However, we advise to keep the table in perfect conditions and make a correct use of it. Unplug the table if you will not use it for a long period of time.

# Technical service



Only a technician authorised by Weelko is allowed to perform any kind of repairs on the bed.

Please contact your distributor or point of sales for further information on technical assistance.



# Technical Specifications

CERVIC reference	2241.1.A26 2241.1.A66
Packaging dimensions (cm)	200 x 75 x 56
Gross weight (kg)	71
Net weight (kg)	68
<b>Motors</b>	
Motors	1 (height)
Height (cm)	53 / 86
Backrest angle (°)	0° ~ 75°
Total length (cm)	190
Total width (cm)	70
Thickness of upholstery (cm)	7
Density	25 kg/m3
Upholstery material	PU - white PU- dark grey
<b>Electrical</b>	
Voltage	220-240 V
Frequency	50/60Hz
Duty cycle	Max. 2Min, On; 18Min, OFF
Maximum power	300W
Max. lifting capacity:	175 kg
<b>Controllers</b>	
Hand controller	HC001
Foot controller	FC001

## Product dimensions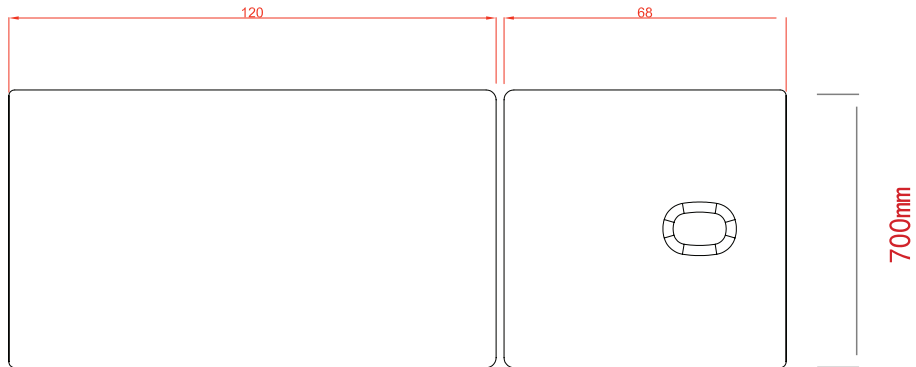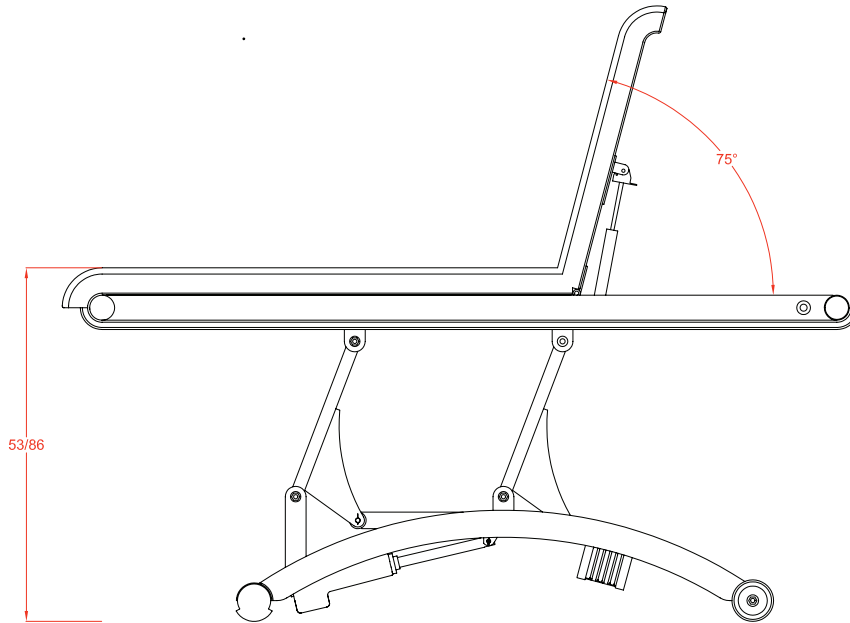

# Backrest dimensions and breather hole

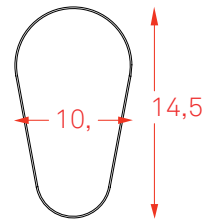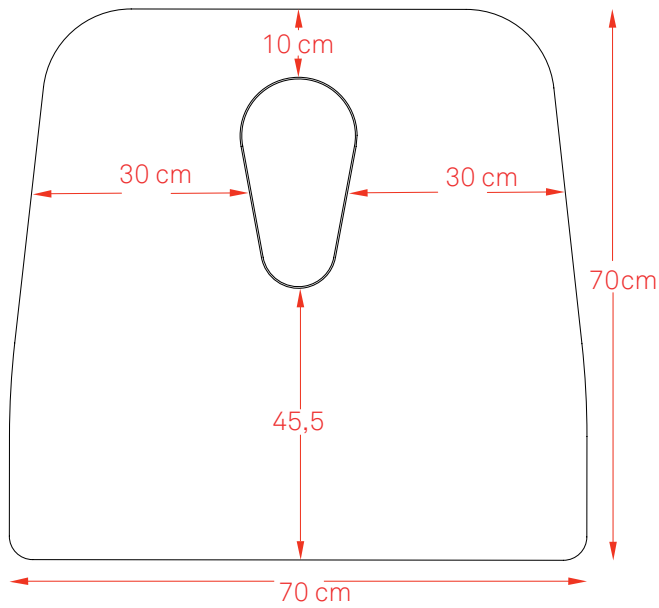Pol. Ind. La Vallan  
Camí de Garreta, 92  
08600 Berga  
Barcelona - Spain  
T. +34 93 824 90 20  
E. info@weelko.com

Our products are subject to  
continuous technical improvements.

Weelko reserves the right to  
change or modify this manual and  
its contents without prior notice.

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