

INDEX

Radiofrequency and sessions	2
Beauty instrument, use and column models	3
Face and body applicators & technical specifications	4
Website information	5

Radiofrequency with **MultiEquipment** by Weelko

¿Loose or lax skin? ¿Ageing? ¿Wrinkles and fine lines? ¿Alteraciones pigmentarias? ¿Dehydration? ¿Sagging? ¿Bags? ¿Uneven skin texture and glow? ¿Cellulite?

Rejuvenate your skin with **MultiEquipment**, F-333

Radiofrequency instrument used in aesthetic facial treatments. Ideal for firming and contouring skin. The equipment applies heat onto the body which stimulates collagen fibres and contributes to eliminate sagging areas. Equipped with 2 handles and 7 heads.

Discover everything here inside.



Radiofrequency

About radiofrequency

It is a non-invasive beauty technique used for skin tightening. Es una técnica de rejuvenecimiento facial que aplica calor en los estratos más profundos de la dermis. The radiofrequency instrument by Weelko is used mainly for facial treatments and in very localised body parts.

The radiofrequency device by Weelko uses MONOPOLAR delivery for skin rejuvenation. It is based on applying a single electrode to the treated area whereas the return plate is placed in contact with a large body surface relatively far removed so that the current goes deeply through the body. As a result, the tissue in the treated area is heated rather deeply and intensely.

What does it consist of?

The radiofrequency gently delivers heat energy in the treated area while acting on skin tissues and boosting blood flow and collagen and elastin production. When the inner layers are heated, the body responds by producing more collagen as it perceives there to be an injury. The body's response to this false perception is to produce collagen, which is its natural reparative response. It also prompts a tightening of the elastin fibres. Both these reactions result in a tighter and firmer skin.

Which are the effects?

- Stimulates blood flow
- Stimulates the body to produce collagen and elastine
- Improves skin elasticity

What is it appropriate for?

- Fine lines
- Flaccidity
- Countouring jaws and chins.
- Softening ridles and lines around the eyes, mouth and forehead.
- Improving flaccid skin under the chin.
- Localised adiposity.

Which are the contraindications?

- Dermal alterations.
- Pregnancy and breast feeding women.
- Severe heart disease.
- Blood coagulation.
- Diseases of connective and muscular tissue .
- Cancer.
- People with pacemakers.
- Metal plates or pins in the treatment area.
- Recent collagen implants.

Sessions

A standard treatment of radiofrequency consists of 4-10 sessions. The duration of each session takes 10 minutes for facial treatments and 30 minutes for body treatments. It is advisable to leave a 15-day interval between sessions.

The side effects of the radiofrequency are:

- Redding and swelling after the session (these results are temporary but they appear quite often).
- Slight equimosis in areas of capilar fragility (not so frequently).



Beauty instrument, F-333:



Brief instructions of use:

- Read carefully the user manual to correctly connect the cables and accessories.
- Apply some conductive gel in the area to be treated.
- Switch on the device with POWER button.
- Press SELECT to choose the desired handle (face or body).
- Place the return metal plate in contact with a large body area. This must be close to the area to be treated, either in facial or body treatments.
- Choose the right-side electrode according to the area to be treated.
- Press START/PAUSE to start the treatment.
- Select the type of wave according to the patient needs. It is recommended to use first the non-continuous wave (indicated for sensitive skin). Then, the wave can be changed according to the professional opinion and patient comfort.
- Put the electrodes onto the skin by making some circle constant movements. Do not leave the electrode fix on an area (risk of burns).
- Once the treatment has been finished press "START/PAUSE".

Beauty column version:

The radiofrequency device is a stand-alone table-top instrument. However, it can also be placed in Weelko's **MULTILAB** trolley or combined with a second beauty device in **K-PSULE** column.



Facial treatment:



Facial handle



Ceramic heads for facial treatments

Body treatment:



Body handle



Ceramic heads for body treatments

IMPORTANT: Place the return metal plate in contact with a large body area. This must be close to the area to be treated, either in facial or body treatments.



Technical specifications:

Dimensions and weight	Packing size (cm)	49 x 45 x 19,5 cm
	Product size (cm) W x H x D	33,5 x 9,5 x 25,5
	Gross weight (kg)	5
	Net weight (kg)	4,5
Technical specifications	Voltage	220V—240V
	Frequency	50 Hz/60Hz
Maximum working power		150 Watts
Working frequency		0,3 MHz (300kHz)

Spare parts:

No.	PRODUCT	REFERENCE
1	Set of 7 ceramic heads	40024
2	Face handle	40019
3	Body handle	40020



Visit our **TECHNICAL INFO** and download all the technical and sales information that you may need. Pictures, dossiers, technical sheets, etc. Do not miss them.

Dossiers | Technical Zoom

Weekko

Products

Beauty

Spa

Physiotherapy

Podiatry

News

Downloads



Turn on the **WEELKO TV** in weekko.com. The best way to learn how to use our equipment. Because a picture is worth a hundred words!



weelko.com

Barcelona
Pol. Ind. La Valldan
Camí de Garreta,92
08600 Berga
Barcelona – Spain
T. +34 93 824 90 20
E. info@weelko.com